

Manual ECG Leads

GENERAL DESCRIPTION

The ECG Cables Series is recommended for its compatibility with various kinds of hospital equipments.

The exact constructions of the cables keep the same isolated from environment.

WARNING

The following factors can reduce the cable yield

- excessive movement
- electrosurgical interference
- bad connection cable
- incorrect type of cable

CLEANING INSTRUCTIONS

Clean the cable with an H₂O solution and a neutral detergent.

Don't submerge the cable.

DESINFECTION

We recommend ethylene oxide.

Oxygen water at 3vol, isopropanol at 70%, ethanol at 70%, CIDEX, solution 1:10 of sodium hypochlorite can be used.

WARRANTY

All cables have a 1 year of full warranty.

The warranty does not include bad treatment or misuse.

The useful life of the cables depends of its use and treatment.

PRECAUTIONS

Please clean the device before use. *Read DESINFECTION*

The cable can be scratched when pulled, misuse can cause the cable to break.

Avoid contact with alcohol, detergent or another alkaline substance.

If you use with ether, acetone or ester the cable will lose its flexibility

Please store cables in a 25° environment. The temperature should not exceed 50°C.

THE CABLES MUST BE USED BY QUALIFIED PERSONNEL